MEDISANA®

home of wellness

DE Wärmeunterbett HU 650 / HU 655

GB Heated underblanket HU 650 / HU 655

FR Couverture chauffante HU 650 / HU 655

IT Materasso termico HU 650 / HU 655

ES Manta bajera térmica HU 650 / HU 655

PT Capa de aquecimento HU 650 / HU 655

NL Warmteonderbed HU 650 / HU 655

FI Lämpöpatja HU 650 / HU 655

SE Värmemadrass HU 650 / HU 655

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
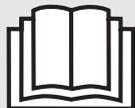
IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

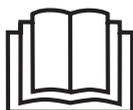Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Protection category II

LOT

Lot number

Manufacturer

Do not puncture the heated underblanket!

Do not use the heated underblanket when it is folded!

Only use the heated underblanket indoors!

**The heated underblanket may be washed by hand according to the instructions on the label!
Unplug the control switch unit from the heating underblanket!**

Do not chlorinate!

Do not dry the heated underblanket in a tumble drier!

Do not iron the heated underblanket!

Do not dry clean!

1.1 Safety Information

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Check the electric underblanket regularly for any signs of wear or damage. If any signs occur, it must be returned and inspected by the manufacturer or an authorised service centre before you use it again.
- Only use the blanket for the purpose described in these Instruction manual.
- Do not use the heated underblanket with small children or persons who are disabled, in need of care or insensitive to heat.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- Four fastening straps are located on the lengths of the electric underblanket. Use these straps to attach the electric underblanket to the mattress using a suitable band or cord. Make sure that the electric underblanket is firmly strapped to the mattress in both directions and that it will not crease while it is being used. Check it is in the correct position before going to bed.
- During use, the control switch and lead must not be placed on or under the blanket or be otherwise covered in any way.
- Before using it on an adjustable bed, make sure that the power cable and the underblanket do not get caught in the hinges or get crumpled when making adjustments.
- No safety pins or other pointed or sharp objects should be attached to the underblanket or stuck into it.
- Never leave the blanket on when unattended.
- The device is only for private use and is not to be used in hospitals.
- Never handle a blanket that has fallen into water. Unplug it from the mains outlet immediately.
- Do not use the blanket in a damp or humid environment or if the blanket itself is damp. You may only use it again once it is completely dry.
- Never expose the control switch or leads to moisture of any kind.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or twist the blanket by the mains lead and do not allow the lead to become tangled or snagged.
- Do not repair the underblanket yourself in the event of damage or malfunction. Return the faulty heated underblanket to your dealer. Only have repairs carried out by an authorized specialist dealer or suitably qualified personnel.
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Let the heated underblanket cool down completely before folding and storing it. Do not fold it too

- Should you have any concerns about health issues, consult your doctor before using the blanket.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

2 Useful Information

Thank you very much

Thank you for your confidence in us and congratulations!

You have purchased the heated underblanket **HU 650 / HU 655**, a quality product by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA** heated underblanket, we recommend that you read the following operating and maintenance instructions carefully.

2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA HU 650 / HU 655** heated underblanket
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your supplier without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

3.1 Use

With the **MEDISANA HU 650 / HU 655** heated underblankets you can gently warm and relax your whole body. Using the blanket stimulates the circulation in areas of tense muscle and promotes a sense of well being after a strenuous day. The heated underblanket is warmer around the feet and provides a pleasant temperature in bed, allowing you to relax and get to sleep quickly. Spread the blanket out flat on top of the mattress. Pull the edges like a fitted sheet over the mattress or other suitable undersurface. Make sure that the electric underblanket is firmly strapped to the mattress in both directions and that it will not crease while it is being used. Spread a bedsheet on top of it. This allows the underblanket to release its heat in the best way possible. Check it is in the correct position before going to bed!

The blanket may only be used when it is spread out flat. The heated underblanket features an electric temperature control that regulates the temperature according to the chosen setting.

3.2 Operation

We recommend switching on the heated underblanket 30 minutes before going to bed.

Plug the blanket into the mains outlet and move the selector switch from position **0** to position **1**. The green power LED will light up. You will feel the blanket gradually warming up. This will be even more noticeable if it is covered with a quilt. If you require a higher temperature setting, move the switch to one of the higher settings, position **2**, **3** or up to the maximum heat setting, position **4**. Should the blanket get too warm for you, slide the switch back to a lower setting.

To switch off the blanket, return the switch to position **0**. The green power LED will go out, indicating that the unit is switched off.

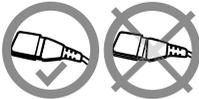
The heated underblanket is designed in such a way that settings **1** and **2** are suitable for continuous use while you sleep. Settings **3** and **4** are best suited to pre-warming the bed.

The blanket will switch off automatically after approx. 3 hours of continuous use. To switch it back on, first move the switch back to position **0**, then back to the desired heat setting.

The unit is fitted with an overheat cut-off, which also switches the unit off automatically if there is any risk of it overheating. Always allow the unit to cool down fully before you use it again.

Unplug the blanket from the mains outlet when it is no longer in use.

4.1 Cleaning and maintenance

Correct

- Before washing the heated underblanket, unplug it from the mains outlet and let it cool down.
- Unplug the control switch unit from the heated underblanket using the connector on the underside.
- The heated underblanket may be washed by hand according to the instructions on the label. Never immerse the control switch unit in water as moisture must not be allowed to get into the unit.
- Never use aggressive cleaning agents or strong brushes.
- Only use the unit again when it is completely dry.
- To dry, lay the heated underblanket flat on a moisture-absorbent towel or mat. When the heated underblanket is completely dry, it should be reconnected to the control switch unit in the correct manner. The heated underblanket may only be operated with its special control unit.
- When storing the heated underblanket, allow it to cool down before folding. Store the heated underblanket in a clean and dry place. When storing, do not place any objects on the heated underblanket to prevent buckling.

4.2 Disposal

Do not dispose this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary.

To return your used device, please use the return and collection system or contact the retailer where the product was purchased. They can take this product for environment safe recycling.

4.3 Technical specifications

Name and model	: MEDISANA HU 650 / HU 655 heated underblanket
Power supply	: 220-240 V~ 50 Hz
Heat output	: approx. 100 W
Switching levels	: 0 - 1 - 2 -3 - 4
Automatic switch off	: after approx. 3 hours
Dimensions	: approx. 90-100 x 200 cm
Weight	: approx. 1.4 kg HU 650 / approx. 1.6 kg HU 655
Mains lead length	: approx. 2.35 m
Operating conditions	: only use dry
Storage conditions	: clean and dry
Item number	: 61210 HU 650 61215 HU 655
EAN number	: 40 15588 61210 7 HU 650 40 15588 61215 2 HU 655

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

**Warranty and
repair terms**

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Jagenbergstraße 19
41468 NEUSS
GERMANY

E-Mail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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**MEDISANA AG**