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EN Safety instructions

- Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The device is intended for indoor private use only and should not be employed for commercial purposes.
- Only use the device for the according the instructions for use. The guarantee is invalidated if the device is misused.
- Avoid contact with sharp or pointed objects.
- Do not use this device if you are pregnant.
- Persons with a pacemaker, artificial joints or electronic implants should always consult a doctor before using the device.
- You must not use the device if you suffer from one or more of the following diseases or medical conditions: Circulatory problems, varicose veins, open wounds, diabetes, fever, heart problems, bruises, skin tears or other skin injuries, phlebitis, tumours, thrombosis, osteoporosis, spinal fractures or slipped discs.
- Do not use the appliance on the following parts of the body: Head, face, front of neck, chest, abdomen, bones or joints (e.g. elbows, knees, etc.), kidney and genital area.
- Excessive pressure when massaging with the device can lead to injuries!
- Special care is necessary when the device is used on, by or near children, sick and helpless persons.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- If you feel any pain from use of this device, or find the massage unpleasant, stop the treatment and consult your doctor.
- Keep the appliance away from heat, hot surfaces, moisture and liquids. Never touch the appliance with wet hands or when standing in water. Do not reach for a unit that has fallen into water.
- Do not cover the device when it is switched on. Never use it under blankets or pillows. There is a risk of fire, electric shock and injury.
- Never store or use the appliance near an electric stove or other heat sources.
- Do not sit or stand on the unit.
- Do not use the appliance together with other appliances.
- Do not use the appliance before going to bed. The massage has a stimulating effect and can delay falling asleep.
- Do not fall asleep while using the massager on yourself. Using it on the same spot for too long can cause discomfort and burns.
- Do not repair the device yourself in case of a malfunction. Repairs may only be carried out by an authorized dealer or an appropriately qualified person.
- Check the unit for damage before use. A defective unit must not be put into operation!
- Cleaning and user maintenance shall not be made by children without supervision.
- Switch off the device after each use.
- Do not use the device immediately after eating or drinking alcohol or when you are tired.
- Do not drop the appliance or subject it to strong shocks.
- Do not apply the appliance while operating vehicles or machinery.
- Make sure that the cable does not become a tripping hazard. It must not be kinked, pinched or twisted. If the cable is defective, it must not be used any longer.
- We recommend that you fully charge the device every 3 months at the latest to avoid deep discharge or damage to the battery.
- This appliance contains batteries that are non-replaceable.

① Handle

② Function button (On/Off/Intensity)

③ Select direction of rotation

④ Function LEDs

⑤ Massage rollers

⑥ Connection for charging cable (behind protective cap)

Charging the battery

Fully charge the battery before using it for the first time. The charging time is 3.5 hours. With a fully charged battery, the device can be used for approx. 3.5 to 5 hours. To charge, connect the supplied USB cable to the device connector ⑥ and a suitable power source (5V DC, 2A). If the connection is correct (charging process), the LED ring ④ pulsates.

The more closed the LED ring, the higher the charge level of the battery.

- NOTES**
- The unit cannot be used during the charging process!

The service centre address is shown on the attached leaflet.

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