MEDISANA®

- DE Handmassagegerät HM 858**
GB Handheld massager HM 858
FR Appareil de massage manuel HM 858
IT Massaggiatore manuale HM 858
ES Aparato de masaje manual HM 858
PT Massajador manual HM 858
NL Handmassageapparaat HM 858
FI Käsihierontalaite HM 858
SE Handmassageapparat HM 858
GR Συσκευή μασάζ χειρός HM 858

Art.-Nr. 88292

Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Istruzioni per l'uso
Instrucciones de manejo

Bitte sorgfältig lesen!
Please read carefully!
A lire attentivement s.v.p.!
Da leggere con attenzione!
¡Por favor lea con cuidado!

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes provide useful additional information on the installation or operation.

Protection category II

LOT

LOT number

Manufacturer

about the power supply

- Before connecting the device to your power supply, make sure that it is switched off and that the electrical voltage specified on the device's type label matches that of your mains power.
- Keep the power cord and the device itself away from heat, hot surfaces, damp and liquids. Avoid touching the plug or power switch with wet or even damp hands, or when you are standing in water.
- If the device falls into water, do not try to pick it up. Instead pull the plug out immediately.
- Always make sure that the plug is easily accessible when the device is connected.
- Always switch off the device immediately after using it and disconnect the plug from the socket.
- Never pull on the power cord to disconnect the appliance from the mains. Always pull out the mains plug!
- Do not carry, pull or rotate the device by its power cord.
- If the supply cord is damaged, it must be replaced by **MEDISANA**, its service agent or similarly qualified persons in order to avoid a hazard.
- Make sure at all times that the power cord does not represent a tripping hazard. The cord should not be kinked, pinched or twisted.

for specific groups of people

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children should be supervised to ensure that they do not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.

- The surface of the appliance is hot. People who are insensitive to heat must be careful when using the appliance.
- Massages using the hand-held massage device are not a substitute for treatments administered by doctors or physiotherapists.
- Do not use this device if you are pregnant.
- People fitted with a pacemaker, artificial joints or electronic implants should in all cases consult a doctor before using the device.
- You should not use this device if you are affected by one or more of the following illnesses and conditions: circulatory disorders, varicose veins, open wounds, bruises, swellings, cracked skin, skin rashes, phlebitis or thrombosis.
- Do not use the massage device on parts of the body that are numb or insensitive to heat, pressure or pain, or only do so after proper consultation with your doctor.
- Never use the device where there is unexplained pain in the stomach or calves.
- If you feel any pain from use of this device, or find the massage unpleasant, stop the treatment and consult your doctor.

before using the device

- Check the device and its power cord and plug carefully for any damage before each use. The device should not be used if it is in any way defective.
- Do not use the device if there is any visible damage to the device itself or to the power cord, if the device is not working flawlessly or if it has been dropped or has fallen into water.
- The device is intended for private use only and should not be employed for commercial purposes.
- Use the massage device only for the purpose specified in the instructions for use.
- The guarantee is invalidated if the device is misused.
- While using the red light function, do not place the massager on the same part of the body for too long (max. 1 minute), as this could cause skin rashes or burns.

- Keep the device away from rain and damp. It should not be operated outdoors.

- It is vital to avoid placing, using or storing the device above containers holding liquids, particularly where the device could fall or be knocked into a bathtub or shower tray.
- Do not use the device whilst in the bath or the shower.
- Do not use the device underneath bedding or cushions. Under no circumstances should you cover the device during use. Overheating may lead to burns, electric shock or personal injuries.
- Do not fall asleep while using the massager on yourself. Using the massager in the same place for a long time can cause complaints and burns.
- Avoid using the device directly on the head, bones or joints, on the front of the throat, on or near the carotid artery, in the genital area or around the kidneys.
- Do not use the device for more than twenty minutes maximum at a time, and allow it to cool down for about thirty minutes before using it again.
- It is best not to use this device directly before retiring - the massage that it delivers has a stimulating effect and may delay sleep.

concerning maintenance and cleaning

- You should not carry out any work on the device, except for cleaning. To avoid possible injury, do not attempt to repair the device yourself. Contact the service point for all repairs.
- Before cleaning the device, switch it off, unplug it from the socket and allow it to cool. Never immerse the device in water or other liquids.
- Read the information in the section “Cleaning and maintenance”.
- In case of malfunctions, do not repair the device yourself. This would automatically invalidate the guarantee. The service point alone should be charged with any repair work.

Thank you very much

Thank you for your confidence in us and congratulations on your purchase! By purchasing the **Handheld massager HM 858** you have acquired a quality product by **MEDISANA**. This unit is intended for body massage with and without heat treatment.

To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** massage device for a long time to come, we recommend that you read carefully the following information about the use and care of the device.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point. The following parts are included:

- 1 **MEDISANA** Handheld massager **HM 858**
- 2 Sets of massage attachments for acupressure and point massage
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

2.2 What is massage?

Massage is one of the oldest methods of treatment and an important part of physiotherapy. The mechanical treatment of the outer tissues of the body relaxes tense muscles, improves blood circulation and supports the regeneration of tissues. Massage is stimulating, relaxing and helps to eliminate pain of the nerves and muscles. Massage may also positively effect the vascular system, the inner organs and metabolism.

There are five classic methods of massage: stroke massage, kneading, tapping and vibrating massage as well as rubbing and slapping. Furthermore, there are other important massage techniques such as manual lymphatic drainage, reflexology, underwater pressure massage, brush massage and finger pressure massage techniques (shiatsu, acupressure).

2.3 How does the HM 858 massager work?

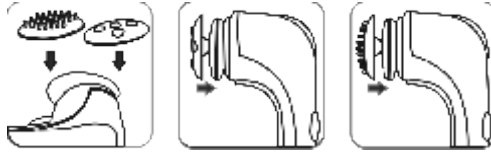
The **MEDISANA HM 858** massager utilizes what is known as vibration massage, whereby the massage head is subjected to high-frequency oscillation, giving rise to a vibrating motion, which is transferred to the parts of the body being massaged. It also features an optional red light warming function, which can be activated either independently or together with the massage.

This special form of heat treatment accelerates the positive effects of the massage. When this is activated, the massage head will warm up slightly, but not actually get hot.

The unique combination of intensive massage and heat treatment allows the **HM 858** massager to offer effective and relaxing therapy.

3.1 Getting started

Ensure that the appliance is switched off (function selector switch ② is in the lowest position) before plugging into the mains. You may use the device directly or you may attach a pair of the massage attachments ⑤ onto the massage heads with red light ④ :

Then plug into an appropriate socket.

3.2 Starting the Massager

Use your thumb to gently push the selector switch ② upwards until it clicks into place and you can feel the massage head vibrate.

The switch features 3 positions:

1. Down position: OFF
2. Middle position: Only massage
3. Up position: Heat and massage

3.3 The massage

- You can massage yourself or a partner. Switch the unit on as described above and apply to the part of body to be massaged. Do not use pressure; just place it lightly on the body. The massage effect is brought about by the vibration of the unit alone. Simultaneous use of the red light function improves the effectiveness of the massage. Do not concentrate on one side of the body for too long (max. 3 minutes); rather change the area of massage frequently.
- You can control the massage frequency by turning the massage speed selector ①. Turn it in clockwise direction to speed up the massage and in anti-clockwise direction to slow down the massage speed.
- By choosing the direction of the massage you can decide on the effect: a massage towards the heart relaxes, a massage away from the heart stimulates.
- For a full body massage, please refer to "Safety Information".
- Each session should not exceed a duration of 15 - 20 minutes. The device will switch off automatically after 20 minutes. To ensure positive results, please use the massager regularly.
- The application should feel pleasant. In case you experience pain or find the massage unpleasant, please stop the session and consult your doctor.
- After treatment, switch the massage device off, pull out the mains plug and let the device cool down.

3.4 Switching off the unit

Switch off the unit by returning the function selector switch ② to its original position. When you have finished using the unit, unplug it from the mains outlet.

3.5 Overheating protection

The device features an overheating protection, which will shut down the device if operated too long. If the device switches off automatically, please disconnect the mains plug and wait for at least 5 minutes, before you use it again.

4 Miscellaneous

4.1 Cleaning and maintenance

- Before cleaning the unit, make sure that it is switched off and unplugged from the mains outlet. Let it cool down.
- The device should be cleaned using a soft, slightly dampened cloth, slightly moistened with soapy water.
- Do not use harsh cleaning agents or stiff brushes.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

4.2 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

4.3 Technical specifications

Name and model:	MEDISANA Handheld massager HM 858
Power supply:	220 - 240V~ 50/60Hz
Power consumption:	25 Watts
Short operation:	20 minutes
Dimensions (l x w x h):	approx. 32.5 x 17 x 14.5 cm
Weight:	approx. 1.4 kg
Operating conditions:	do not use under high temperature and/or moist environment such as bathroom
Storage conditions:	in a cool, dry place
Article No.:	88292
EAN Code:	40 15588 88292 0

In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Medisana GmbH

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41468 NEUSS

Germany

eMail: info@medisana.de

Internet: www.medisana.com

The service centre address is shown on the attached leaflet.

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