## DE Körperanalysewaage BS 444

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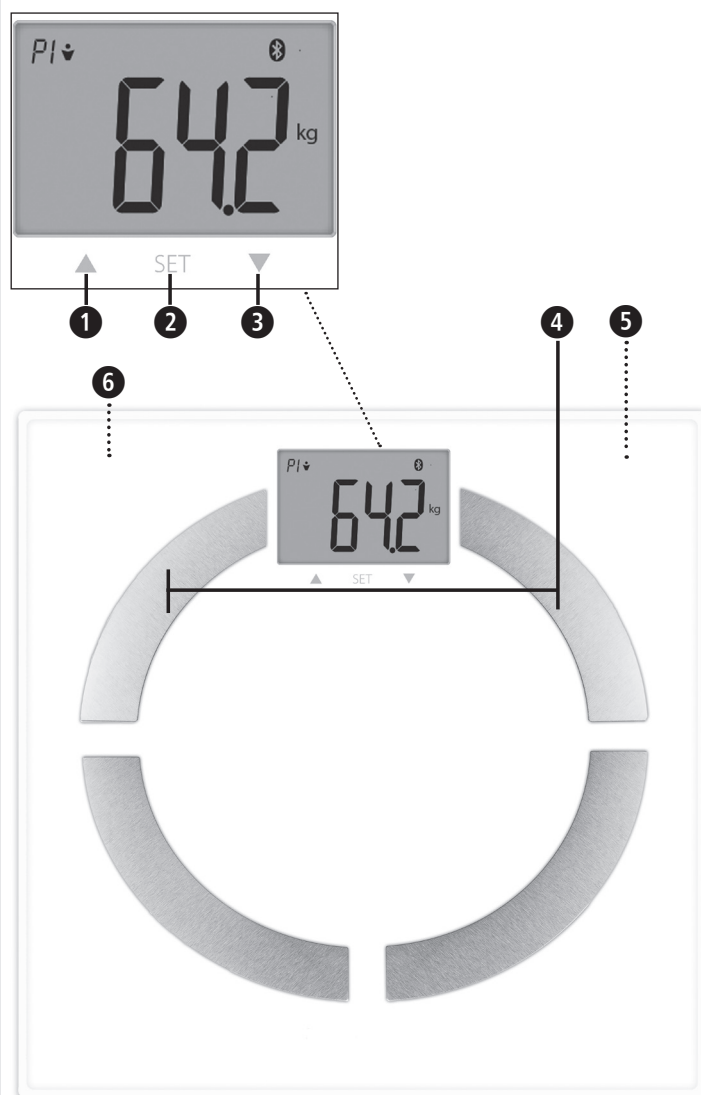
## GB Body analysis scale BS 444

- 1 ▲ Up button
- 2 SET button
- 3 ▼ Down button
- 4 Electrodes
- 5 Battery compartment (on the underside)
- 6 Switch for weight units (on the underside)

### Legend:

This instruction manual belongs to this device. The instruction manual includes important information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the device.

### WARNING!

These warnings must be followed to prevent possible injury to the user.

### LOT

Batch number

Manufacturer

## GB Safety instructions

**Please read these instructions for use carefully before using the device, in particular the safety instructions. Keep the instructions in a safe place for future reference. If you pass the device on to a third party, please also include these instructions for use.**

- This scale is not suitable for people with pacemakers or other medical implants. The indication of body fat may be inaccurate in the event of diabetes or other medical/physical conditions. The same applies to highly trained sportsmen.
- This device is not suitable for pregnant women!
- Any treatment or diet for overweight or underweight requires qualified advice from appropriate specialists (doctor, dietician). The values determined with the scale can be of help.
- Risk of tilting! Never place the scales on an uneven surface. Never step onto just one corner of the scale. While weighing, never stand on one side or on the edge of the scale.
- Danger of slipping! Never step onto the scale with wet feet. Never step onto the scale while wearing socks.
- Use the device only for its intended purpose, as directed by the user instructions. In the event of misuse, all guarantees become null and void.
- The scale is manufactured for home use. It is not intended for commercial use in hospitals or other medical facilities.
- Do not use the device if it does not work properly, if it has been dropped, dropped into water or damaged.
- The scale has a measuring range up to 180 kg/396 lbs. Do not overload the scale.
- Place the scale on a firm level surface. Soft, uneven surfaces are unsuitable for measurement and lead to incorrect results.
- Place the scale in a location where neither extreme temperatures nor extreme humidity are present.
- Keep the scale away from water.
- Please handle the scale with care. Avoid shocks and vibrations of the scale. Do not drop it.
- Stand carefully on the scale. Do not hop or jump on the tread surface, as this may damage the weighing mechanism.
- Do not attempt to disassemble the device, otherwise the warranty will become void. This device does not contain any parts that can be serviced or replaced by the user.
- In the event of malfunctions, do not proceed with any unauthorised repair of the device. Otherwise, this will render any warranty claims void. Have repairs performed only by an authorised service centre.

### Battery safety instructions

- Do not take the batteries apart!
- If required, clean the battery and device contacts before insertion!
- Removed spent batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! In the event of contact with battery acid, immediately rinse the affected area with plenty of clean water and consult a doctor immediately!
- If a battery is swallowed, consult a doctor immediately!
- Insert the batteries correctly, paying attention to the polarity!
- Keep the battery compartment closed properly!
- If the device is not used for an extended period of time, remove the batteries!
- Keep the batteries out of the reach of children!
- Do not re-charge the batteries! There is a risk of explosion!
- Do not short-circuit the device! There is a risk of explosion!
- Do not throw into a fire! There is a risk of explosion!
- Store unused batteries in the packaging and not near metal objects to avoid a short circuit!

### Scope of delivery

First, check whether the device is complete and is not damaged. If in doubt, do not operate the device and contact your service centre.

The contents include:

- 1 medisana Body analysis scale **BS 444**
- 3 Batteries (Type AAA, 1.5 V)
- 1 Instruction manual

The packaging is recyclable or can be recycled into raw materials. Please dispose of unwanted packaging material properly. If you notice any transport damage while unpacking, please contact your supplier immediately.

### WARNING

**Make sure that children do not play with the packaging films. Choking hazard!**

### Requirements for correct measurement results

- Always weigh yourself in the same conditions.
  - Measure at the same time of day.
  - Stand upright and calm.
  - It is best to take the measurement in the morning, after showering or bathing, with dry feet.
- With the results of the measurement, when carried out consistently, a reliable control of the development of the body weight is possible.

### Initialise scale

Make sure that the scale is on firm and level ground. If you want to weigh yourself, but have moved the scale immediately before, you must first initialise the scale. Briefly press the middle of the scale surface with your foot. The display will show "0.0". When the scale has turned off, it is ready for the "step-on" function. If the scale has not been moved beforehand, this process is omitted.

### Connect with the VitaDock+ App

Download the VitaDock+ App free from the AppStore or the Google Play Store.

Activate Bluetooth® on your iOS or Android device. Start the App. Tap the Bluetooth® icon to go to the device selection. Scroll down, until the scale is displayed in the device list. Tap the arrow next to the illustration to start the setup process.

### Simple weighing

1. Stand on the scale and stand still. The device automatically switches itself on.
2. Your weight is measured, flashes twice and then appears constantly.
3. Step down off the scale.

**Optional:** Transfer the measured weight\* to the VitaDock + App.

Prerequisites:

1. Bluetooth is activated on your Android or iOS device
2. The VitaDock+ App has been started.
3. The scale is already listed in the device list in the App.

The setup is described under "Connecting to the VitaDock + App".

\* No body analysis data are transmitted here and no weight history is created.

### Body analysis

The scale can store the data of 8 persons: gender, age, height.

1. Briefly press the middle of the scale surface with your foot. The display will show "0.0 kg".
2. Press the SET button 2. The memory location flashes in the display.
3. Now you can select the desired memory location for your user profile (0 - 8) by pressing button 1 or button 3.

NOTE: If you use the scale in guest mode "P0", there will be no data storage. Transmission of the measured values via Bluetooth® is not possible in this mode.

4. Press the SET button 2 to save the setting.
5. Continue to adjust the gender, athlete mode, size, and age settings.
6. After you have set your personal data, the display shows "0.0". To finish the scale settings, simply stand barefoot on the scale.
7. The display shows the following measured values:

- Weight • Proportion of body fat • Proportion of body water • Proportion of muscle mass
- Bone weight • BMR (calorie requirement) • BMI

When a complete measurement could be taken, the data are then sent to the VitaDock+ App via Bluetooth.

### Automatic user selection

Automatic user selection based on the last measured weight: Stand barefoot on the scale. This automatically selects your storage space and then starts a new measurement. If the correct storage space is not displayed automatically, use "▲" 1 or "▼" 3 to select the correct number (P1 to P8) and confirm the selection with "SET" 2.

### Change the weight unit

Briefly press the middle of the scale surface. The display shows «0.0». You can now select the desired units (kg - lb - st) with the button on the back of the scale.

### Error messages

---- = The scale is overloaded. **LO** = The battery is weak and needs to be replaced. **ERR2** = No body analysis values can be determined. **P1P2** or other memory combinations = The scale recognises users already stored based on their weight. In this case, two users have a similar weight. To select the correct user press the up key for the left value (example "P1") and the down key for the right value (example "P2"). If your scale does not work as desired, check the following before contacting the service centre • Check that the battery has been correctly inserted • Check that you have selected the correct weight unit. • Check whether the scale is standing freely on a firm and level surface. It must not touch a wall or another object. • Repeat the weighing process.

### Cleaning and care

Never use aggressive cleaning agents or stiff brushes. Clean the scale only with a soft, slightly damp cloth. Never use harsh detergents or alcohol. No water should enter the device.

Only use the device again when it is completely dry.

### Instructions for disposal

Do not throw this device away with ordinary household waste. Metals are reusable or can be recycled into raw materials. Dispose of a device that is no longer needed at a collection point in your town/city so that it can be disposed of in an environmentally friendly way. Please contact your local authority or retailer with regard to disposal procedures. Remove the battery before you dispose of the device. Do not dispose of spent batteries in household waste, rather in special waste or at a battery collection point at your retailer!

### Technical data

**Name and model:** medisana Body analysis scale BS 444

**Power supply:** 4.5 V (3x 1.5 V, AAA)

**Measuring range:** up to 180 kg/ 396 lb

**Division:** 0.1 kg/0.2 lb

**Dimensions:** 300 x 300 x 23 mm

**Weight:** 1.63 kg

**Item No.:** 40444

**EAN No.:** 4015588 40444 3

**Frequency band:** 2402 ~ 2480 MHz

**Maximum radio-frequency power transmitted in the frequency band:** -1.0dBm

In the course of constant product improvements, we reserve the right to make technical and design changes.

The current version of these user instructions can be found at [www.medisana.com](http://www.medisana.com)

### Warranty and repair conditions

In case of warranty please contact your specialist shop or the service centre directly. If you need to return the device, please indicate the defect and enclose a copy of the purchase receipt. The following warranty conditions apply:

1. All **medisana** products are guaranteed for 3 years from the date of purchase. The date of sale is to be proven in case of warranty by the purchase receipt or invoice.
2. Defects due to material or manufacturing defects shall be eliminated free of charge within the warranty period.
3. A warranty service does not extend the warranty period for either the equipment or any replacement parts.
4. The following are excluded from the guarantee:
  - a. any damage caused by improper handling, e.g. by non-observance of the instruction manual.
  - b. Damage due to repair or intervention by the purchaser or unauthorised third parties.
  - c. Transport damage that has occurred on the way from the manufacturer to the consumer or when sending it to the service centre.
  - d. Spare parts that are subject to normal wear and tear, such as batteries etc.
5. Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty case.

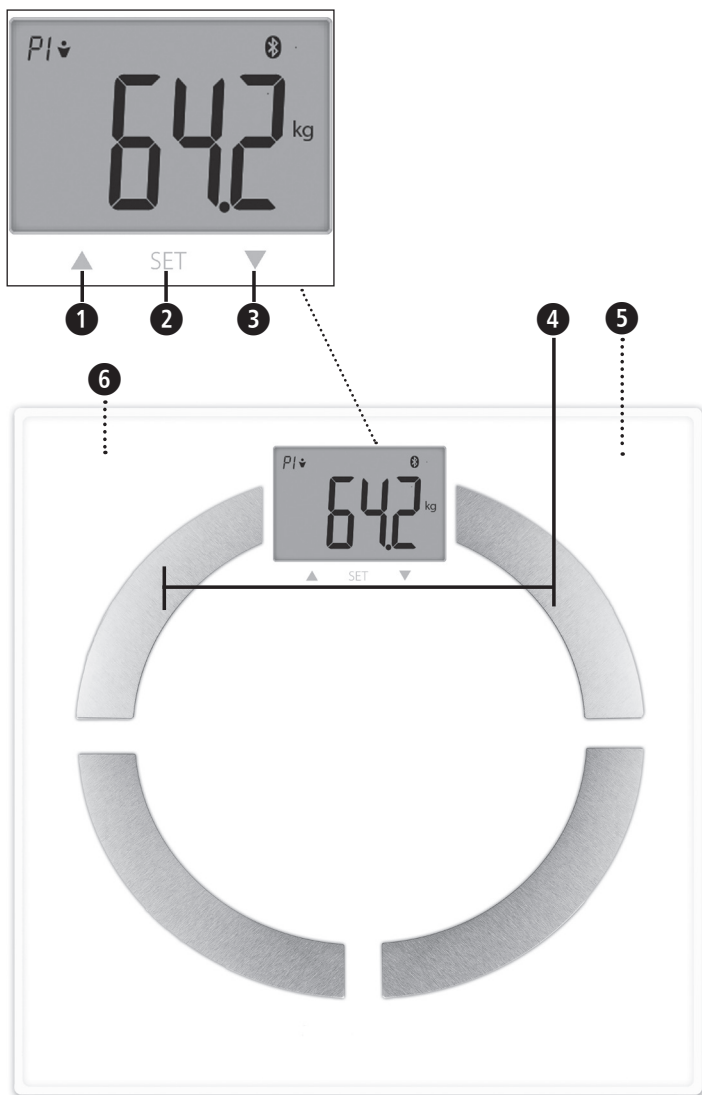
**medisana GmbH**  
Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

The service address can be found on the separate attachment sheet.

We hereby declare that bathroom scales model BS 444, art. 40444 complies with the fundamental requirements of the European RED 2014/53/EU. You can request the complete EC Conformity Declaration from medisana GmbH, Carl-Schurz-Str. 2, 41460 Neuss, Germany, or you can also download it from the medisana homepage ([www.medisana.com/bs444connect](http://www.medisana.com/bs444connect)).

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