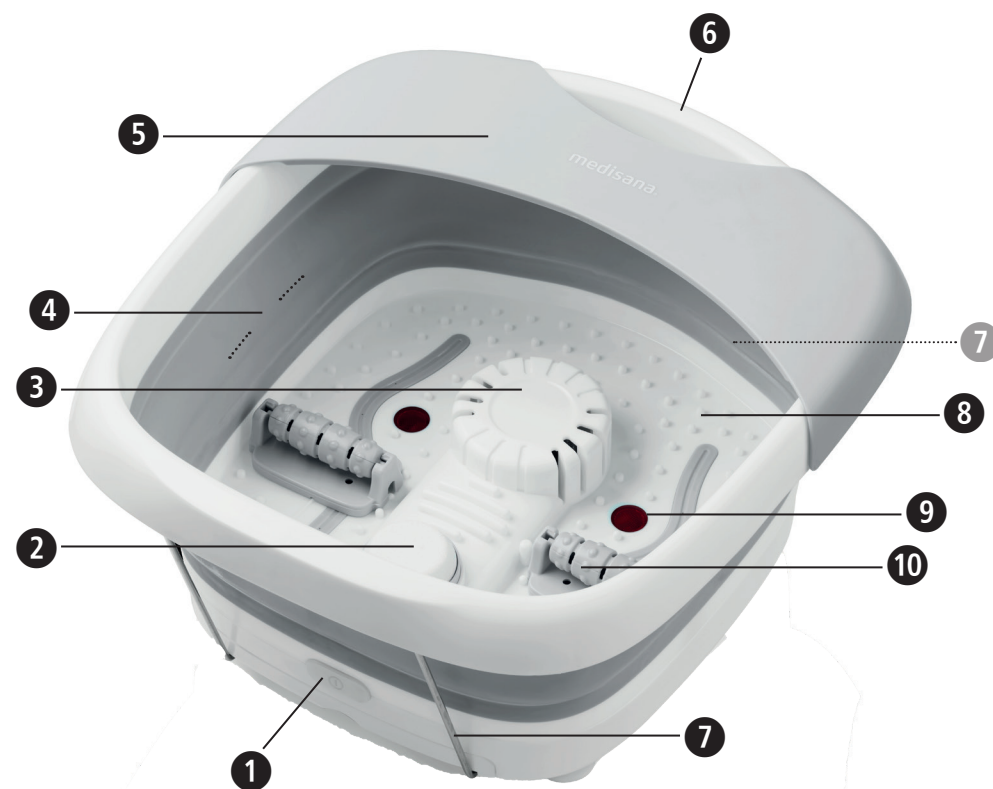
medisana®

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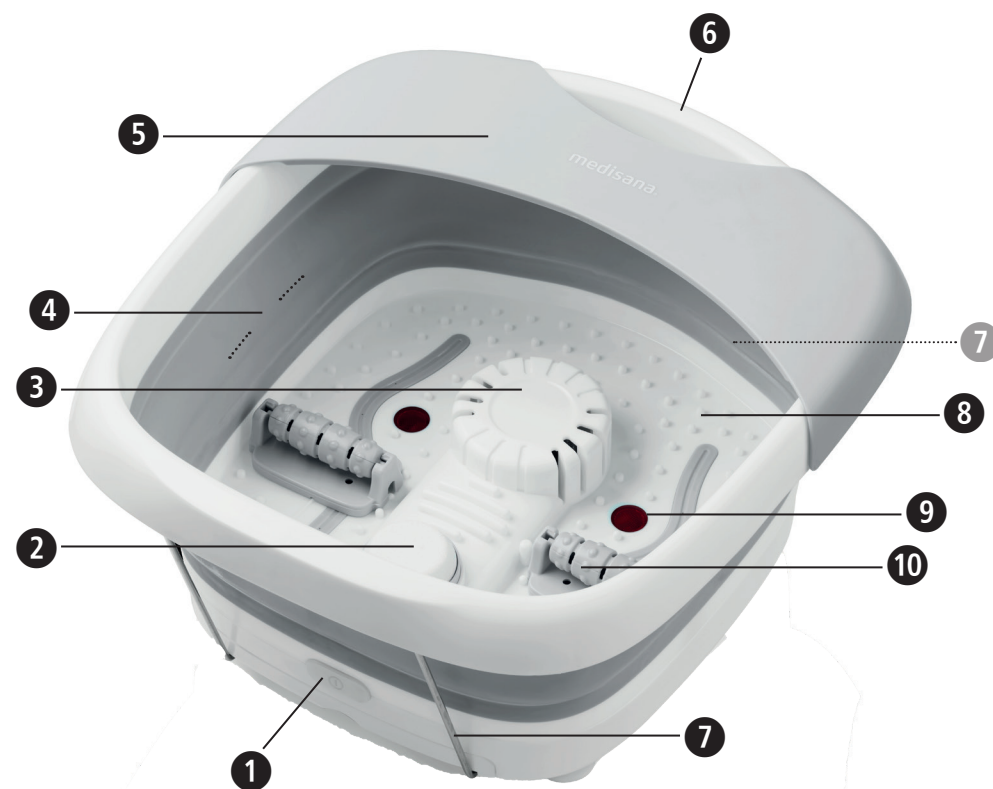
medisana®

Foot Spa

Instruction manual - Please read carefully!

88381_88392_FS_886_05/2021_V06.11

Not to be used by very young children (0-3 years)!

Explanation of symbols

IMPORTANT

Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

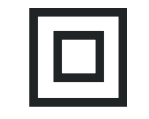
NOTE

These notes give you useful additional information on the installation and operation.

Only use the unit in enclosed spaces!

Protection category II

LOT

Lot number

Manufacturer

Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

Power supply

- Before connecting the device to the power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only connect the mains plug to the wall socket when the device is switched off.
- If the power cable is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Do not touch the mains plug when you are standing in water and always take hold of the plug with dry hands.
- Do not take hold of a device if it falls into water. Unplug the mains cable immediately.
- Keep the device and its power cable away from hot surfaces.
- Avoid the contact of the device with pointed or sharp items.
- Never carry, pull or turn the device by the electricity cable and do not squeeze the cable.
- Position the mains cable so as to avoid the danger of falling.
- Switch off all functions after use and remove the plug from the mains supply.

For special people

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The device must not be used by young children over the age of 3 years, unless the function button has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the device properly.
- Should you experience medical complaints during use talk to your doctor. Should this be the case, stop using the device immediately.
- If you have medical problems with your feet, vein diseases or diabetes, talk to your doctor before using the device. Should you have inexplicable pain or swellings in your legs or feet or have injured a muscle, please consult your doctor before use.
- We advise consultation with your doctor if you are pregnant.
- Stop a massage if you experience pain during the application or swellings crop up after use.
- The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

Operating the device

- Only use the device for its intended use as described in the instruction manual. If misused, all guarantee rights shall become null and void.
- The device is not determined for commercial use, but only for domestic use.

- Do not use the device outdoors or in damp rooms.
- Make sure that you use your foot spa on a safe and even surface that is not sensitive to moisture.
- Only use the device with water and not with other liquids.
- Only use appropriate bath additives - do not use foaming bath additives or bath salts or oils.
- Do not stand in the device as it is not designed to take the burden of full body weight.
- Never cover the device during operation and do not use it under blankets or cushions.
- If water leaks from the appliance, the appliance should no longer be used.

Maintenance and cleaning

- In case of a malfunction, do not try to repair the device yourself. Not only does the warranty expire, but serious danger can result (fire, electric shock, injuries). Only authorised service offices should carry out a repair.
- Cleaning and user maintenance shall not be made by children without supervision.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.

Device and controls

- 1 Function button:
 - 1x = BUBBLE ;
 - 2x = BUBBLE & HEAT & RED LIGHT ;
 - 3x = OFF ;
- 2 Pedicure attachment place
- 3 Massage rollers (removable)
- 4 Filling line (max)
- 5 Removable splashguard
- 6 Trough for pouring out the water
- 7 Set-up bracket (front and back)
- 8 Massage nubs
- 9 Redlight section
- 10 Massage rollers (removable)

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. If in doubt, do not use it. Send it to a service point.

The following parts are included:

- 1 medisana Foot Spa
- 3 exchangeable pedicure attachments (callus remover, brush, massager)
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that the packing is kept away from the reach of children!
Risk of suffocation!

Thank you

for your confidence in us and congratulations!
You can use the **medisana Foot Spa** to give your feet a treat and to relax after a hard day. You can take approx. 10 to 15 minutes for a footbath with a massage, once or twice per day. However you should make sure, that the device has cooled down fully after each use before it is used again!

CAUTION

Make sure that you use your **medisana Foot Spa** on a safe and even surface that is not sensitive to moisture. Never use any foaming bath additives or bath salts or oil.
Never use the appliance without water!

Set up the appliance

Press the bottom of the foot spa down while holding the top ring of the unit. The foot spa is fully unfolded when the total height is approx. 42 cm. Pull the two support brackets 7 forwards and backwards out of the appliance and fold them upwards. Press the two stirrups under the ring so that the stirrups are anchored there.

Use

Fill the unit with warm or cold water up to the marked filling line on the inside 4. Place the unit on a level surface and plug the mains plug into a socket. Now sit comfortably in front of the appliance and set the desired function with the function key 1.

Press the function key once to switch on an invigorating bubble bath.

Press the function key 1 once more to activate the heat function (water is heated up to 45 °C) and the red light.

Pressing the key again deactivates all functions.

After use, disconnect the mains plug from the socket and empty the foot spa via the trough 6.

Additional massage effect

Use the massage rollers 10 and the massage nubs 8 on the bottom of the foot spa to specifically intensify the massage effect on the feet.

Additional pedicure attachments

With the **medisana Foot Spa** you additionally have the opportunity to use one out of three included pedicure attachments. Place the callus remover, the brush or the massage attachment on the pedicure attachment application place 2. Now rub your foot with smooth pressure on the attachment put on the attachment application place 2. Do not use too much pressure. You may vary the application area on your foot by changing the position of your foot on the attachment. With the brush and the massager attachments you can massage the soles of your feet and the callus remover attachment is suitable to gently remove hardened skin on your feet.

Cleaning and care

- Before cleaning the device, make sure that it is turned off and the mains plug is removed from the socket.
- Pour the water out and clean the device with a cloth when it has cooled off. You may use a suitable disinfectant.
- Please do not use any aggressive substances, e.g. scouring chemicals or thinners when cleaning. These could damage the surface.
- Store the device in a dry, cool place.
- Wind-up the electricity lead carefully to avoid it parting.

Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

Technical specifications

Name and model: **medisana Foot Spa**

Type FS 886

Power supply: 220-240 V~ 50-60 Hz

Power consumption: 500 W

Dimensions: approx. 42.2 x 38.5 x 20.1 cm

Weight: approx. 2.16 kg

Article no.: 88381/88392

EAN Code: 40 15588 88381 1

40 15588 88392 7

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Guarantee and repair conditions

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH, Carl-Schurz-Str. 2,
41460 NEUSS, Germany

The service centre address is shown on the attached leaflet.

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