## DE Digitale Personenwaage PS 500

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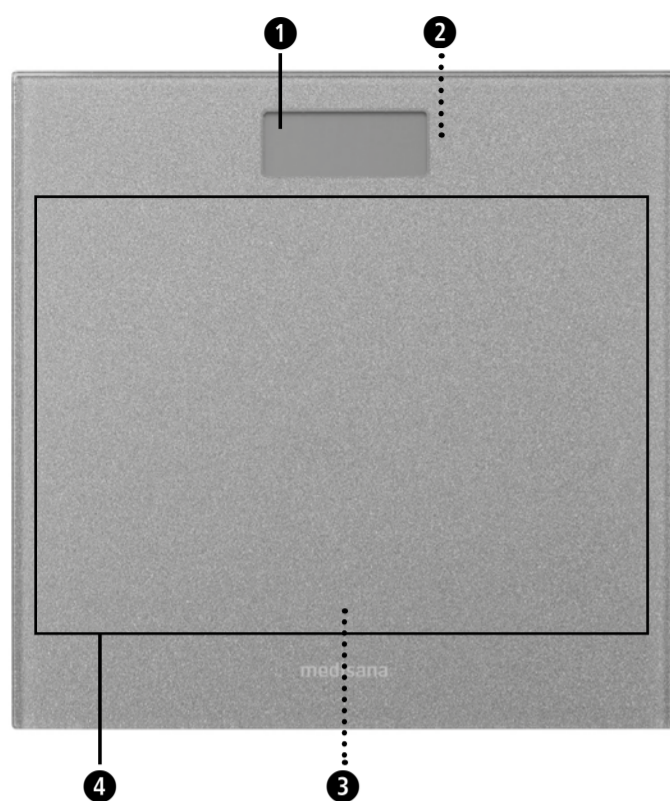
## GB Digital Scales PS 500

- ① Display
- ② Unit key (on underside) for kilograms (kg) pounds (lb) stones (st)
- ③ Battery Compartment (on underside)
- ④ Standing area

### Explanation of symbols:

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING!**  
These warning notes must be observed to prevent any injury to the user.

### LOT

LOT number

Manufacturer

## GB Safety Information

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

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### BATTERY SAFETY INSTRUCTIONS

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately! Increased risk of leakage!
- Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the batteries correctly, observing the polarity!
- Always keep the battery compartment well closed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!

### Scope of supply

Please check first of all that the unit is complete and is not damaged in any way. If in doubt, do not use the appliance. Send it to a service point.

The following parts are included:

- 1 **MEDISANA Digital Scales PS 500**
- 1 Battery (Typ CR2032, 3 V)
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

### Prerequisites for correct measuring results

- Always weigh yourself under the same conditions.
- Always weigh yourself at the same time of day.
- Stand up straight and stand still.
- Weigh yourself preferably in the morning, after showering or bathing, and with dry feet.

It is possible to obtain a reliable indication of body weight development from the results if measurements are carried out consistently.

### Initialise the scales

Make sure the scales are on a firm, level surface. If you only want to weigh yourself but have moved the scales just before, you must first initialise the scales. To do this, press your foot on the middle of the surface of the scales. "0.0 kg" appears in the display. When the scales switch off, they are ready for the step-on function. This procedure is not necessary if you have not moved the scales.

### Weighing

1. Step onto the scales and stand still. The device switches on automatically and the display briefly shows ,0.0 kg'.
2. Your weight is measured, flashes twice and is then finally displayed.
3. Step down from the scales. The scales switch off automatically after 8 seconds.

### Change the unit of weight

Press your foot on the middle of the surface of the scales shortly to activate the device. Then press button ② until the unit you wish is shown.

### Insert/replace battery

The insulating strip may be pulled out of the battery compartment ③ on the underside of the unit before the scale can be operated. Replace the batterie if the battery change symbol "Lo" appears in the display or if nothing appears in the display after you have stepped on the device. Please note polarity (positive pole upwards; facing towards the lid).

### Error messages

**Err** = The scale is overloaded. **LO** = The battery is low and must be replaced.

If your scales do not function to your satisfaction, check the following before contacting a service centre: • Check whether the batteries have been inserted correctly. • Check whether the correct unit of weight has been selected. • Check whether the scales are free-standing on a solid and even surface. They must not be touching a wall or any other object. • Weigh yourself again.

### Care and maintenance

Never use aggressive cleaning agents or stiff brushes. Clean the scale using a soft and lightly moistened cloth. Never use abrasive cleaning agents or alcohol. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.

### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

### Technical specifications

**Name and model:** MEDISANA Digital scales PS 500

**Voltage supply:** 3 V (1x 3 V, CR2032)

**Measuring range:** up to 180 kg / 396 lb / 28 st 4 lb

**Graduation:** 0.1 kg/0.2 lb/ 1/4 lb

**Dimensions:** 30 \* 30 \* 2.1 cm

**Weight:** 1.5 kg

**Item number:** 40490, 40491

**EAN number:** 4015588 40490 0, 4015588 40491 7

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

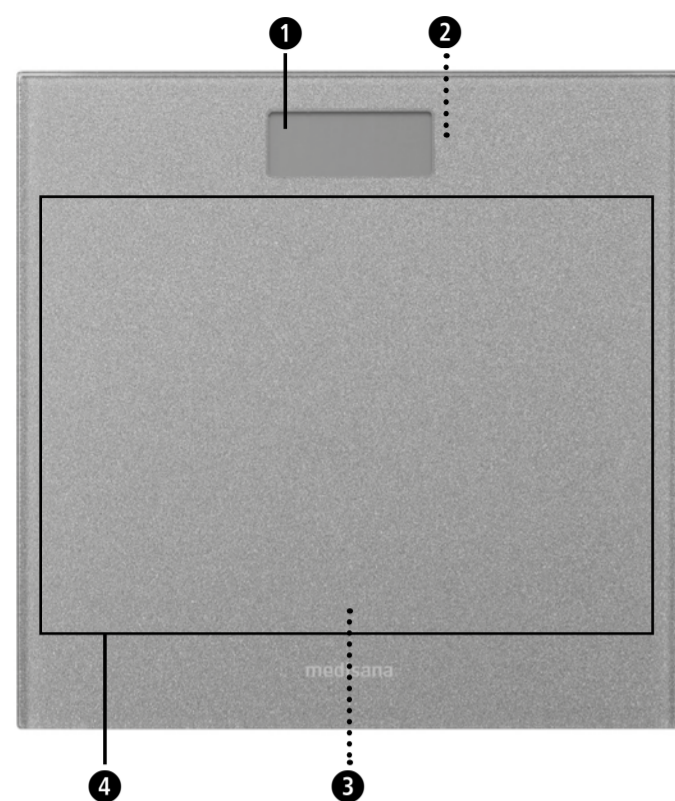
### Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for MEDISANA products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**MEDISANA GmbH**  
Jagenbergstr. 19  
41468 NEUSS  
GERMANY

The service centre address is shown on the attached leaflet.

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