MEDISANA®

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Refer to instruction manual recommended.

These warning notes must be observed to prevent any injury to the user.

These notes must be observed to prevent any damage to the device.

These notes give you useful additional information on the installation or operation.

These notes give you useful additional information on the installation or operation.

Ingress protection rating

Classification: Type BF applied part

Lot number

Serial number

Manufacturer

Date of manufacture

This product must not be disposed of together with domestic waste.

Do not dispose of old batteries with your household waste!

CE symbol (conformity symbol)

Refer to instruction manual recommended.

These notes give you useful additional information on the installation or operation.

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GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.
Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Use the unit only as intended in accordance with the directions in the instruction manual within the specified environmental conditions.
- Using the unit for any other purpose will void your guarantee.
- The unit is not designed for commercial use.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children shall not play with the appliance.
- The unit is not suited for children under the age of 3.
- If you have a pacemaker, artificial limbs or an electronic implant, please consult your physician before using the unit.
- If you experience discomfort during the use, such as allergic reactions or other complaints, stop using the device immediately and contact your physician.
- The unit is not recommended for persons who suffer from epilepsy or heart disease.
- Use the unit only on dry and combed hair.
- Do not use the unit on dry, cracked or wounded skin.
- **When using the electric lice comb, avoid direct contact with your ears, eyes and mouth. Touching the metal teeth can inflict a painful, yet harmless, electrical charge.**
- The comb has rounded pearl-shaped tips on every second tooth ➋. Make sure that no teeth are missing or damaged before you use the unit. Otherwise the lice comb is no longer useable.
- In the event of a malfunction, do not attempt to repair the unit on your own; this will void your guarantee. Have all repairs carried out by an authorised service centre.
- Be aware that unauthorised conversion and / or modification of the product is prohibited. In case of unauthorised conversion and / or modification the warranty cover will be rendered null and void.
- You are only permitted to clean the unit.
- Keep the unit away from hot surfaces and guard it against exposure to direct sunlight.
- The unit may not be operated in the vicinity of high-frequency electromagnetic emitters.
- Protect the unit from moisture. Should liquid get into the unit, remove the batteries immediately and refrain from using the unit. In such a case, contact your dealer or notify us directly. Our address and phone number are located in this manual.
- Do not use the unit if the housing or comb teeth are damaged, if the unit does not function properly or if it has fallen down or fallen into water.
- Prevent accidents or damage to the unit by keeping the unit away from clothing, bristles, wires, strings, shoelaces, threads etc. when it is switched on.
- The unit may occasionally leave your scalp tingling. This is perfectly normal and is in no way dangerous.
- Clean your lice comb after every use, in particular when more than one person is being treated, to avoid the risk of transmitting lice.
- Always switch off the unit when not in use.
- When not in use, always put the protective cap back ➌ on the unit.

Special features of the MEDISANA electric lice comb LC 860
The **MEDISANA** electric lice comb **LC 860** effectively eliminates head lice without having to resort to chemical means. It works by creating an electric charge between the teeth that kills off the lice and their eggs on contact. The electric charge is so small that users will not notice it; it is absolutely safe to use. Comb activity is indicated by an audible signal. The unit is powered by conventional AA batteries. The **MEDISANA** electric lice comb **LC 860** is suitable for adults and children over the age of 3. The comb can also be applied to household pets, such as dogs and cats.

Operating
Inserting / removing the battery

Remove the battery compartment lid ➋ and insert the included battery (type AA LR6). Observe the proper polarity (+/-) of the battery as indicated on the battery itself and on the inside of the battery compartment. Replace the battery compartment lid ➋. Follow the same procedure when replacing a battery. We recommend that you do not use rechargeable batteries. Remove the battery from the unit if you do not plan on using it again anytime in the near future.

Troubleshooting
Problem: Device does not work.
Solution: Exchange the battery. If the device still does not work, contact the customer service.

Disposal
This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Technical specifications

Name and model: **MEDISANA** electric lice comb **LC 860**
Power supply: 1,5 V battery (type AA, LR 6)
Material: Housing: ABS
Teeth: Copper, chrome-plated
Tip of teeth: epoxy resin
Operating conditions: +5°C to +37°C,
15 - 90 % rel. humidity,
atmospheric pressure
700 hPa to 1060 hPa
Transport/Storage conditions: -25°C to +70°C,
15 - 90 % rel. humidity,
atmospheric pressure
700 hPa to 1060 hPa
Dimensions L x W x H: approx. 193 x 66 x 24 mm
Weight: approx. 71 g incl. battery
The device is designed for a usage period of 5 years.
Battery life: 5 years.
41017
EAN number: 40 15588 41017 8

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under www.medisana.com

For service, accessories and spare parts, please

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