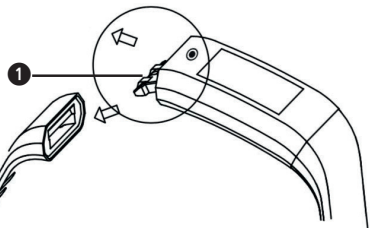
medisana®

DE ViFit Run

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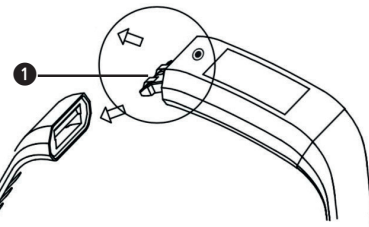
medisana®

EN ViFit Run

- 1 USB port (charge battery)
- 2 Touch sensor

! WARNING! Risk of swallowing by small children! Children are not allowed to play with the device. Keep it away from children.

! WARNING! Danger of fire/explosion or chemical burns! The item contains a lithium-ion battery. This must not be removed, taken apart, thrown into a fire or short circuited.

! CAUTION! Possible property damage! Do not drop the device and protect it from impact. Do not expose the device to extreme temperatures or extreme temperature fluctuations. Protect the device from direct sunlight and dust. When cleaning, do not use any strong chemicals, or aggressive or foaming cleaning materials. Protect the display from hard, scratching objects.

! CAUTION! Do not dispose of the device in the household waste! If you have any questions, please contact your municipal waste disposal authority. Do not dispose of spent batteries in domestic waste, rather in special waste or at a battery collection point at your retailer!

The packaging is recyclable or can be recycled into raw materials. Please dispose of unwanted packaging material properly.

CE We hereby declare that the ViFit Run Activity Tracker, Item no. 79492 complies with the basic requirements of European Directive 2014/53/EU. The complete EC Declaration of Conformity can be requested from medisana GmbH, Carl-Schurz-Str. 2, 41460 Neuss, Germany or can be downloaded from the medisana homepage. Subject to technical changes in the course of constant product development.

medisana GmbH
Carl-Schurz-Str. 2
41460 NEUSS
GERMANY
Email: info@medisana.de
Internet: www.medisana.de
The service address can be found on the separate attachment sheet.

Activate device

Before use, connect the device to a USB charger in order to charge it for activation. The progress bar changes from empty to full, to indicate the complete activation as below.

! After activation, the battery is charged.

! If the battery charge of the device falls below 10%, the adjacent symbol will be ticked to remind you to charge the battery.

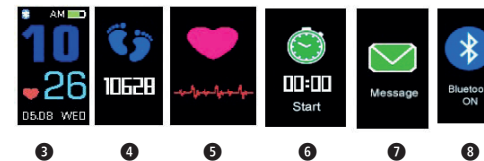
Connect with the VitaDock+ App

Download the VitaDock+ App free from the AppStore or the Google Play Store. Activate Bluetooth® on your iOS or Android device. Start the App. Tap the Bluetooth® icon to go to the device selection.

Scroll down until ViFit Run is displayed in the device list. Tap the arrow next to the illustration to start the setup process.

Accessing functions

Briefly touch the touch sensor to run through the display of each function. You can always return to the respective initial display by pressing and holding!

3 Date and time display

Press and hold: display current weather conditions

In the weather conditions display: press briefly to display the current temperature and the temperature range.

Note: The feature can only be accessed if the device has been connected to the VitaDock+ App.

4 Step counter

Press and hold: Open the display with details on sports performance.

Press briefly: Switch between the performance data (calories burned, distance, active time and current percentage of the daily target).

5 Heart rate

Press and hold (3 seconds): Start the heart rate measurement.

Note: For the best result, the bracelet must be worn tightly. If this is not the case, the device will vibrate gently and prompt you to check the fit of the device on your wrist.

During the heart rate measurement, 2 to 5 different training zones are displayed, depending on the rate of change of the heart rate.

Press the touch sensor again for 3 seconds to stop the heart rate measurement and return to the main screen.

6 Stopwatch and reminder function

Press and hold: Start the stopwatch

Press briefly: Stop or resume timing.

End the stopwatch function: Stop the stopwatch first, then press and hold on the touchscreen.

You can set the reminder functions (tablet intake, movement reminder) via the VitaDock+ App.

7 Messages (only with existing Bluetooth® connection to your iOS or Android device)

The device displays messages immediately when they are received.

The messages are stored under the menu item "Message". In this way, the device stores up to 20 messages. Thereafter, the oldest of the messages stored is deleted or overwritten.

Messages from the following sources are displayed: SMS, WhatsApp, Facebook Messenger

Calls: When the notification of an incoming call is displayed, you can reject the call by pressing and holding the touch sensor. If you press the touch sensor briefly, the notification including vibration will stop, but the call will not be rejected on your smartphone.

8 Bluetooth® activate/deactivate

Press and hold: Bluetooth® activate/deactivate

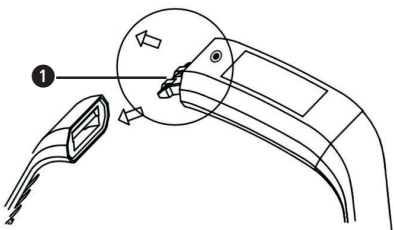
Sleep mode (Sleep tracker)

The device automatically detects if you are asleep and records whether your sleep is restless or quiet. However, you can also manually set a period in which the sleep mode should be activated via the VitaDock+ App.

Technical data

Battery capacity: 110 mAh
Charging time: 1.5 hours
Battery life: 7-10 days (standby)

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