## Activity Tracker **ViFit connect MX3**

<b>DE</b>	Gebrauchsanweisung.....	2
<b>GB</b>	Instruction Manual.....	14
<b>FR</b>	Mode d'emploi.....	26
<b>ES</b>	Instrucciones de manejo.....	38
<b>NL</b>	Gebruiksaanwijzing.....	50

Art. Nr. 79790

## DE - Inhalt

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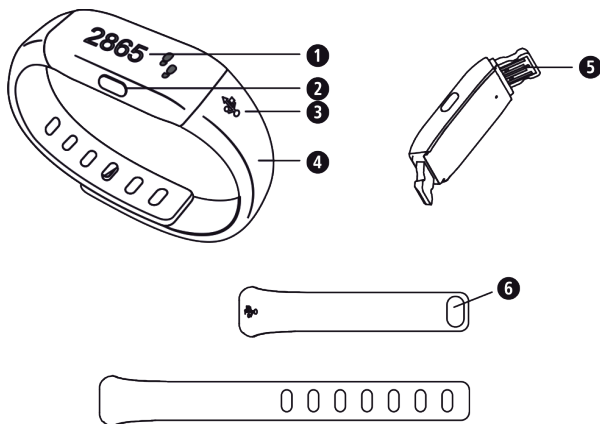
## GB - Contents

Controls.....	14
Before first use.....	15
Activity mode.....	17
Alarm setting.....	20
Exercise reminder setting.....	20
Sleep mode.....	21
Flight mode.....	22
Data transfer to smartphone.....	23
Important notes .....	24

## Controls

- 1 OLED-Display
- 2 Function button
- 3 USB symbol

- 4 Wristband
- 5 USB-port
- 6 Wristband lock button

## Before first use

With the activity tracker **ViFit connect MX3** you may determine the number of steps, the passed distance, burned calories, duration of activity and the degree of achieving motion objective in %, as well as the actual time of day and the date. The device features 2 alarms and an exercise reminder alarm. With the sleep mode the motion activity during sleep and the duration of sleep is recorded.

To charge the battery, connect the USB-port ⑤ of the device to a PC or Laptop. The USB-port is suitable for charging only and not for the transfer of data. The charging state of the built-in battery will be displayed as follows:

Battery empty - please  
charge immediately

Battery  
charging

Battery fully charged -  
Charging process complete

After the charging process, download VitaDock+ app free of charge from AppStore or Google Play Store.

**iOS:** iPhone4S and newer, iPad 3 and newer. **Android:** Devices, which support Google-Android-Version 4.3 and **Bluetooth®** 4.0 technology.

Before first use, the device should be activated from the transport mode by pressing and holding (for approx. 5 seconds) the function button ②.

Now activate **Bluetooth®** on your iOS- or Android device.

## Before first use

Start the app.

Make sure, that tracker and smartphone are switched on and **Bluetooth®** is activated. Select your activity tracker **ViFit connect MX3** in the app under "my devices" in order to pair it. Press and hold the function button ❷ for approx. 10 seconds to pair it with the app. During pairing process, the following symbols appear on the OLED-display ❶:

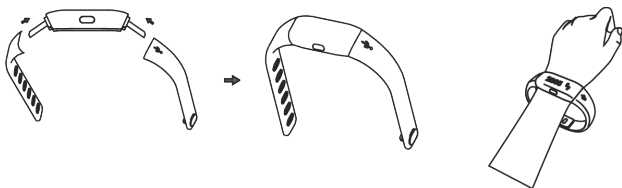
If the devices have been successfully synchronized, "**Success!**" appears in the display. If not, the message "**Fail!**" appears. In this case, check whether **Bluetooth®** is activated correctly, both devices are switched on and the app is correctly installed. Lower the distance between both devices and try a new pairing process.

When the devices are successfully connected, adjust the personal settings within the app. The tracker will be synchronized accordingly. The actual time and date will be automatically transferred to the tracker after establishing a successful connection.

# Activity mode

How to wear the activity tracker:

The wristband ④ is adjustable in size according to your wrist.

Additionally, the wristband may be exchanged - there are 2 different sizes included in the scope of delivery.

You may fix the desired wristband on the device according to the above shown drawings.

If you wear the wristband on the left wrist, the function button ② should point towards your body. If you wear the wristband on the right wrist, the function button ② should point towards your fingers.

# Activity mode

Press the function button **2** shortly to switch the display as follows:

Welcome text only at first switch-on  
(Press and hold button **2** for approx. 5 sec.)

Time of day, Date, Battery state,  
**Bluetooth®**-logo

Step count

Burned calories (kcal)

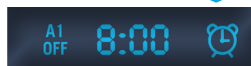
Distance in km

Activity time in hours


Daily target in %

Alarm 1 (**Off** or **On**)

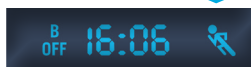Alarm 2 (**Off** or **On**)

Exercise reminder alarm (**Off** or **On**)

## Activity mode

Press the function button **2** to execute the following functions:

- short press => next screen in display  
(see previous page)
- press and hold (for approx. 3 seconds)  
=> Sleep mode
- press and hold longer (for approx. 6 seconds)  
=> Flight mode
- press and hold even more longer (for approx.  
10 Sekunden) => Pairing mode

After putting on the **ViFit connect MX3** correctly (see page 17), it will record your steps. After you have walked 10 steps or more, it will display and count any further steps. This is necessary to exclude possible disturbing factors or convulsions.

The step count will be reset to zero at midnight (0:00 a.m.) each day. You may recall the steps again via the app.

The data is saved on the activity tracker for a maximum of 7 days. If you don't synchronize the tracker with your smartphone during this period, the oldest values will be overwritten.

## Alarm setting

You may set 2 different alarm times (A1 and A2) with time of day and day of the week with the app.

If an alarm will be active within the next 24 hours, the display will show „A1“ resp. „A2“ and „**On**“, the alarm symbol and the set alarm time. Otherwise it will show „**Off**“.

## Exercise reminder setting

For the setting of the exercise reminder the above procedure is almost similar. A programmed exercise reminder will be shown in the display with „**B**“ and „**On**“ or „**Off**“, the set time of the day and the runner symbol.

Each alarm or exercise reminder will make the tracker vibrate upon reaching the set time.

If you want to stop an alarm, just press the function button **2** shortly.

## Sleep mode

In sleep mode, the movement activity during sleep and the duration of sleep is tracked. You can find more evaluation possibilities in the VitaDock+ app.

To activate sleep mode, press and hold the function button **2** for approx. 3 seconds. The activation will be confirmed by a short vibration.

The activity tracker will also be wearred on the wrist during sleep. When sleep mode is active, a small moon symbol appears on the right display edge as in the following example:

To exit sleep mode, press and hold the function button **2** again for approx. 3 seconds. The de-activation will also be confirmed by a short vibration.

## Flight mode

**Bluetooth®** will be active for some seconds with each press on the function button ②. There is a possibility to switch the tracker into a flight mode, in which **Bluetooth®** is completely deactivated and your steps will not be captured. To activate the flight mode, press and hold function button ② for approx. 6 seconds. The following symbols appear in the display:

Afterwards, a display similar to the following example appears on the activity tracker:

To exit flight mode, press and hold the function button ② again for approx. 3 seconds.

The activation and the de-activation of flight mode will be confirmed by a short vibration.

## Data transfer to smartphone

Start the app. If smartphone and tracker have been paired correctly before, the app will recognize the device by shortly pressing the function button **2. Bluetooth®** is then switched on.

Follow the directions given by the app to transfer the data on you tracker to the VitaDock+ app.

# Important notes

**WARNING!** Danger of swallowing by small children!

Children are not allowed to play with this device. Keep children away.

**WARNING!** Danger of fire/explosion or severe burns!

This device contains a lithium-ion-battery. This battery must not be removed, disassembled, thrown in fire or short-circuited.

**ATTENTION!** Possible property damage! Protect the unit against moisture! Keep the unit away from water!

**ATTENTION!** Possible property damage! Never let the device fall and protect it against shocks. Never expose the device to extreme temperatures or strong temperature fluctuations. Also keep the device away from direct sunlight and dust. Never use harsh chemicals, aggressive or abrasive detergents. Protect the display against hard, scraping objects.

**ATTENTION!** Do not dispose of in domestic waste. If you have any further queries, consult your municipal disposal authority.

We hereby declare, that the Activity Tracker ViFit connect MX3, Art. 79790 complies with the fundamental requirements of the european R&TTE Directive 1999/5/EC. You can request the complete EC Conformity Declaration from Medisana AG, Jagenbergstraße 19, 41468 Neuss, Germany, or you can also download it from the Medisana homepage.

Service and further information: [www.medisana.de/ViFit\\_connect\\_MX3](http://www.medisana.de/ViFit_connect_MX3)

We reserve the right to make technical alterations in the course of ongoing product development.

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**MEDISANA AG**