**DE** LCD-Anzeige 2 Verschluss-kappe 4  
 Sensor-spitze 1 ON/OFF-Taste (Start) 3

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**Faking temperatures: fast and error-free!**  
 Fevers are often associated with fatigue, body aches, shivering, and chills. The course of the disease can best be controlled by measuring the fever two to three times a day at intervals of 4 to 6 hours. The measurement should always be performed in the same area of the body and at about the same time for the results to be comparable. The **medisana FTC thermometer** makes it quick and easy to take a current body temperature. Frequent errors such as not leaving the thermometer in place long enough can almost be ruled out. A temperature measurement can be carried out in the following areas: a) under the tongue (oral), b) in the armpit (axillary), c) in the rectum (rectal).

**Measurement**  
 Turn the thermometer on by pressing the Start button 3. All of the symbols will appear for about two seconds, followed by a short beep. The most recent reading will then be briefly displayed on the LCD display 2 (with a small M appearing on the right edge of the display). If the temperature is less than 32°C, you will see "1" and a flashing "C" on the LCD display 2. This means the **medisana FTC thermometer** is ready to use. Position the sensor tip under the tongue, in the armpit, or in the rectum. The thermometer will then automatically start measuring the temperature. During the measurement, the current temperature is displayed continuously. As soon as the measurement is completed, 10 beeps sound and the measured temperature is shown on the display. If the thermometer detects a temperature of more than 37,7 °C, a fever alarm will sound in the form of 10 triple beeps. The thermometer switches off automatically eight minutes after the measurement. To extend battery life, however, you should turn the thermometer off by pressing the Start button 3 once you have read the result.

**Warranty and repair conditions**  
 In case of a warranty claim, please contact your specialist shop or take the thermometer directly to the service point. Should you have to return the device, please state the defect and enclose a copy of the purchase receipt or invoice. The following warranty conditions apply:  
 1. **medisana** products are guaranteed for 3 years from the date of purchase. In case of a warranty claim, the purchase receipt or invoice must be provided as proof of the purchase date.  
 2. Defects due to material or manufacturing defects will be remedied free of charge within the warranty period.  
 3. A warranty service does not extend the warranty period if either the device or for the components replaced.  
 4. The following are excluded from the warranty:  
 a. any damage caused by improper handling, e.g. failure to follow the instructions for use.  
 b. damage due to repair or intervention by the purchaser or unauthorised third parties;  
 c. transport damage that has occurred on the way from the manufacturer to the consumer or when sending it to the service point;  
 d. accessories subject to normal wear.  
 5. Liability for direct or indirect consequential damage caused by the device is also excluded when the damage on the device is recognised as a warranty claim.

**Taking the temperature in different parts of the body**  
**The medisana FTC thermometer** is suitable for taking temperatures under the tongue, in the armpit, or in the rectum. It is completely waterproof. After about 1 minute, 10 short beeps will sound and the measurement result is shown on the LCD display 2. Then the thermometer can be removed. To ensure a safe and accurate measurement, you should consider the following when positioning the thermometer:

1. On **medisana** product is provided a 3 year warranty.
2. Guarantee case through the purchase quality or invoice to be proven.
3. 2 defects due to material or manufacturing defects will be remedied free of charge within the warranty period.
3. Through a warranty service the warranty period is not extended if either the device or for the components replaced.
4. The following are excluded from the warranty:
  - a. any damage caused by improper handling, e.g. failure to follow the instructions for use.
  - b. damage due to repair or intervention by the purchaser or unauthorised third parties;
  - c. transport damage that has occurred on the way from the manufacturer to the consumer or when sending it to the service point;
  - d. accessories subject to normal wear.
5. Liability for direct or indirect consequential damage caused by the device is also excluded when the damage on the device is recognised as a warranty claim.

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**Cleaning and disinfection**  
**The medisana FTC thermometer** is waterproof and easy to clean and disinfect. Before and after each measurement, clean the thermometer with a cloth soaked in a mild alcohol solution. You can also place the thermometer in a suitable disinfectant solution. Your specialist dealer will gladly recommend suitable cleaning products and advise you regarding their use. Do not over boil the thermometer!  
**MEMORY FUNCTION**  
 The **medisana FTC** thermometer has a memory function that allows you to recall the most recent reading. Turn the thermometer on by pressing the Start button 3. All of the symbols of the display will appear for about two seconds, followed by a short beep. The most recent reading will then be briefly displayed on the LCD display 2 (with a small M appearing on the right edge of the display).

**Instructions for disposal**  
 Do not throw this thermometer away in ordinary household waste. Consumers are legally obliged to bring all used batteries, rechargeable or otherwise, whether they contain harmful substances or not, to a collection site in their community so that they can be transferred to the proper channels for disposal. Remove the batteries before disposing of the device. Do not throw used batteries in ordinary household waste, but instead only with other hazardous waste or at a battery collecting station at a specialist shop. Contact your local authority or dealer regarding disposal.  
**Technical information**  
 Name and model: **medisana FTC** Thermometer  
 Voltage supply: Button cell battery 1,5V (Type GP192 or LR41)  
 Range of measurements: 32,0°-42,0°C  
 Measurement accuracy: ± 0,1°C (deviation from the actual body temperature in the measuring range of 35,5°-42,0°C) not extrapolating  
 Operating conditions: 10,0°-40,0°C  
 Storage and transport conditions: -10,0° to +60,0°C  
 Dimensions (L x W x H): approx 10 mm x 128 mm x 19 mm  
 Weight: approx 9,8 g  
 77030  
 EAN Code: 40 15588 77030 2

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