DE Heizkissen

GB Heat Pad

FR Coussin chauffant

IT Termoforo

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PT Almofada térmica

NL Verwarmingskussen

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SE Värmekudde

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Protection category II

LOT

Lot number

Manufacturer

Do not puncture the heated cushion!

Do not use the heated cushion when it is folded!

Not to be used by very young children (0-3 years)!

Only use the heated cushion indoors!

**The heating pad can be washed by hand!
You may wash the cushion cover in a normal wash at max. 30 °C!**

Do not chlorinate!

**Do not dry the heated cushion in a tumble drier!
The cushion cover may not be put in a dryer!**

**Do not iron the heated cushion!
You may iron the cushion cover at a low temperature!**

Do not dry clean!

1 Safety Information

- Carefully check the heat pad for signs of wear and/or damage before each use.
- Do not use it if you notice wear, damage or signs of improper use on the device, the switch or the cables or if the device is not working.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not fold or crease the heat pad during use and storage.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Do not use the device on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The device must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Never use the pad unsupervised.
- The device is only for private use and is not to be used in hospitals.
- Do not fall asleep while the heat pad is switched on.
- Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions. The control switch should not be covered or placed on or below the pad while the unit is in operation.

- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be warmed.
- Never touch a heat pad that has fallen into water. Unplug the unit from the mains outlet immediately.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controls and leads must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- A damaged power supply cable must be replaced with a power cable of the same type.
- Heat pad may only be operated with the control unit (MAG 01HP) supplied.
- When storing the heat pad, allow it to cool down before folding.
- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.
- Avoid bending the pad too much as objects placed on it during storage may cause damage.

Thank you very much

Thank you for your confidence in us and congratulations!
You have purchased the heat pad **HP 605**, a quality product by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA** heat pad **HP 605**, we recommend that you read the following operating and maintenance instructions carefully.

2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA HP 605** Heat Pad with cover
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.
If you notice any transport damage during unpacking, please contact your supplier without delay.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

3 Operating

3.1 Use

You can use the **MEDISANA HP 605** heat pad to gently warm and relax various parts of the body. Using the heat pad stimulates blood circulation in hardened muscles and helps you to relax after a strenuous day.

The heat pad features an electric temperature control that regulates the temperature according to the chosen setting.

Only use the heat pad in the cover supplied with it.

The heat pad features an electronic temperature control that regulates the temperature according to the chosen setting.

Only use the heat pad in the cover supplied with it.

Place the heating pad around your neck with the opening at the front and close the fastener.

3.2 Operation

Plug the unit into the mains outlet and move the slider control from position **0** to position **1**. The power LED will light up in green. After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, move the switch to the next positions, **2**, **3**, or to the maximum heat setting, position **4**. If the pad gets too hot for you, move the slider back to position **3**, **2** or **1**.

After a short while the heating pad cools down noticeably.

To switch the unit off, move the switch back to position **0**. The green power LED will go out, indicating that the unit is switched off. Unplug the unit from the mains outlet.

Move the selector switch to position **1** or **2** before falling asleep or any continuous use. The heat pad will switch off automatically after approx. 90 minutes of continuous use. To switch it back on again, move the slider control to position **0** and then back to the desired heat setting.

Unplug the unit from the mains outlet once you have finished using it.

4 Miscellaneous

4.1 Cleaning and maintenance

Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heating pad is equipped with a detachable cable. Disconnect the plug and remove the cable from the heating pad.
- You may wash the heating pad gently by hand. The best way to do this is to put the heating pad in a bath with lukewarm water and some mild detergent and then gently squeeze it out.
- Rinse the heating pad several times to remove all the detergent.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.
- Only use the appliance once it is completely dry.

Heat pad cover

- Remove the heat pad from its cover.
- Dry clean the removable cover or as detailed in the instructions on the label.

4.2 Disposal

Do not dispose this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary. To return your used device, please use the return and collection system or contact the retailer where the product was purchased. They can take this product for environment safe recycling.

4.3 Technical specifications

Name and model	: MEDISANA HP 605 Heat Pad
Power Supply	: 220-240 V AC 50 Hz
Heat output	: approx. 100 watts
Automatic switch off	: after approx. 90 min
Switching levels	: 0 - 1 - 2 - 3 - 4
Operating conditions	: Only use in dry rooms as described in the operating instructions
Storage conditions	: Laid out flat, clean and dry
Dimensions	: approx. 33 x 40 cm HP 605
Weight	: approx. 480 g HP 605
Mains lead length	: approx. 2.60 m
Article No.	: 61148 HP 605
EAN Code	: 40 15588 61148 3 HP 605

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH

Carl-Schurz-Str. 2

41460 NEUSS

GERMANY

E-Mail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the attached leaflet.

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