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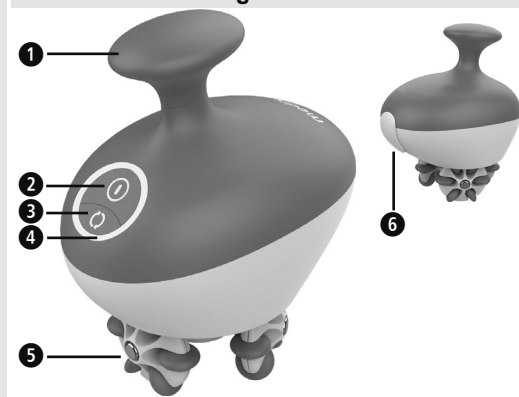
- 1 Handle
- 2 Function button (On/Off/Intensity)
- 3 Select direction of rotation
- 4 Function LEDs
- 5 Massage rollers
- 6 Connection for charging cable (behind protective cap)

Explanation of symbols

IMPORTANT!
Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes provide useful additional information on the installation or operation.

Protection category II

LOT LOT number **IPX4** Splash water provee

Manufacturer

Scope of delivery

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your dealer or your service centre.

The following parts are included:

- 1 **medisana AC 900**
- 1 USB charging cable
- 1 instruction manual

If you notice any transport damage when unpacking, please contact your dealer immediately.

WARNING
Please ensure that the polythene packing is kept away from the reach of children!
Risk of suffocation!

EN Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The device is intended for indoor private use only and should not be employed for commercial purposes.
- Only use the device for the according the instructions for use. The guarantee is invalidated if the device is misused.
- Avoid contact with sharp or pointed objects.
- Do not use this device if you are pregnant.
- Persons with a pacemaker, artificial joints or electronic implants should always consult a doctor before using the device.
- You must not use the device if you suffer from one or more of the following diseases or medical conditions: Circulatory problems, varicose veins, open wounds, diabetes, fever, heart problems, bruises, skin tears or other skin injuries, phlebitis, tumours, thrombosis, osteoporosis, spinal fractures or slipped discs.
- Do not use the appliance on the following parts of the body: Head, face, front of neck, chest, abdomen, bones or joints (e.g. elbows, knees, etc.), kidney and genital area.
- Excessive pressure when massaging with the device can lead to injuries!
- Special care is necessary when the device is used on, by or near children, sick and helpless persons.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- If you feel any pain from use of this device, or find the massage unpleasant, stop the treatment and consult your doctor.
- Keep the appliance away from heat, hot surfaces, moisture and liquids. Never touch the appliance with wet hands or when standing in water. Do not reach for a unit that has fallen into water.
- Do not cover the device when it is switched on. Never use it under blankets or pillows. There is a risk of fire, electric shock and injury.
- Never store or use the appliance near an electric stove or other heat sources.
- Do not sit or stand on the unit.
- Do not use the appliance together with other appliances.
- Do not use the appliance before going to bed. The massage has a stimulating effect and can delay falling asleep.
- Do not fall asleep while using the massager on yourself. Using it on the same spot for too long can cause discomfort and burns.
- Do not repair the device yourself in case of a malfunction. Repairs may only be carried out by an authorized dealer or an appropriately qualified person.
- Check the unit for damage before use. A defective unit must not be put into operation!
- Cleaning and user maintenance shall not be made by children without supervision.
- Switch off the device after each use.
- Do not use the device immediately after eating or drinking alcohol or when you are tired.
- Do not drop the appliance or subject it to strong shocks.

- Do not apply the appliance while operating vehicles or machinery.
- Make sure that the cable does not become a tripping hazard. It must not be kinked, pinched or twisted. If the cable is defective, it must not be used any longer.
- We recommend that you fully charge the device every 3 months at the latest to avoid deep discharge or damage to the battery.
- This appliance contains batteries that are non-replaceable.

Charging the battery

Fully charge the battery before using it for the first time. The charging time is 3.5 hours. With a fully charged battery, the device can be used for approx. 3.5 to 5 hours. To charge, connect the supplied USB cable to the device connector 6 and a suitable power source (5V DC, 2A). If the connection is correct (charging process), the LED ring 4 pulsates. The more closed the LED ring, the higher the charge level of the battery:

0-10% rote LED pulsiert	11-25% LED pul- siert	26-50% LEDs pulsieren	51-75% LEDs pulsieren	76-95% LEDs pulsieren	100%

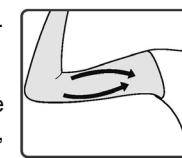
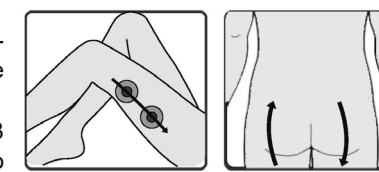
NOTES

- **The unit cannot be used during the charging process!**
- **Do not use any cable other than the one supplied for charging! If you need a replacement, contact the customer service.**

Use

1. Switch on the unit by pressing the function button 2. The device starts to work at low intensity, which is accompanied by the lighting up of half the LED ring 4. If you press function key 2 again, the device switches to high intensity and the full LED ring 4 lights up.
2. You can perform the massage on yourself or on a partner. Place the device on the area of the body to be massaged. Depending on how you feel, you can apply no pressure or light pressure. You can roll along different parts of the body, see „Application examples“.
3. By pressing the button 3 you can change the massage direction.
4. When massaging, do not concentrate on the same part of the body for too long (max. 1 minute), but change the massage area more frequently during the application.
5. An application should not last longer than 15 minutes (after this time the device will automatically switch-off). Use the device regularly so that the positive effect will appear.
6. If you want to switch off the unit manually, press the function button 2 again when unit is in high intensity mode.

Application examples

- Spot massage (hold the device on one spot)
- Move the massager along

NOTES

- **If you find direct application to the skin uncomfortable, you can also place a towel between the skin surface and the device during application.**
- **Application e.g. after bathing increases the circulation-promoting massage effect, as the body is already warm.**

Troubleshooting

If the unit cannot be switched on, check the battery charge level (press and hold function key 2 for 3 seconds). Charge the battery if necessary. If you notice any abnormal behaviour, smells or if the unit still does not work, stop using it and contact the service centre.

Cleaning and maintenance

- For cleaning, make sure that the charging cable is **not** connected and let the unit cool down for at least 10 minutes after use.
- The unit should be cleaned with a soft, slightly damp cloth, possibly with a mild soap solution.
- Never use aggressive cleaning agents, thinners, petrol, strong brushes or similar. The surface of the unit is sensitive.
- The massage rollers 5 can be pulled off and thus cleaned separately, e.g. with water. Then put the cleaned massage rollers back on.
- Only use the unit again when it is completely dry and make sure that no moisture can get into the unit!
- It is best to store the unit in its original packaging and keep it in a clean and dry place out of direct sunlight.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

Technical specifications

Name and model:	medisana Cellulite massager AC 900
Supply voltage:	5V==
Built-in rechargeable battery:	Lithium-ion 2200 mAh
Operating conditions:	0°C to +40°C, 20 to 85 % max. relative humidity
Storage conditions:	-10°C to +60°C, 20 to 85 % max. relative humidity
Dimensions approx.:	85 x 140 x 130 mm
Weight approx.:	345 g
Art. number:	88542
EAN number:	40 15588 88542 6

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

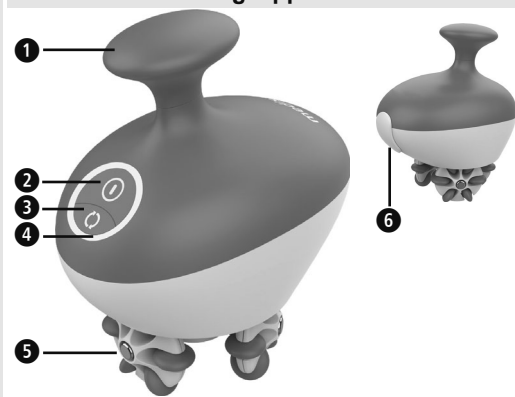
Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana GmbH
Carl-Schurz-Str. 2
41460 NEUSS
GERMANY

The service centre address is shown on the attached leaflet.

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