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GB Instruction manual  
Massage Gun with Hot & Cold function  
MG 600

#### Explanation of symbols

**IMPORTANT**  
Failure to follow these instructions may result in serious injury or damage to the device.

**WARNING**  
These instructions must be followed to prevent possible injury to the user.

**CAUTION**  
These instructions must be followed to prevent possible damage to the device.

**NOTE**  
These instructions provide useful additional information regarding installation and operation.

**LOT** LOT number **SN** Serial number

Manufacturer Date of manufacture

Recycling symbols/codes:  
These are used to provide information about the material and its proper use and recycling.

**WARNING!**  
Please ensure that the polythene packing is kept away from the reach of children!  
Risk of suffocation!

## GB Safety Instructions

Please read these instructions carefully before using the device, in particular the safety instructions. Keep the instructions in a safe place for future reference. If you pass the device on to third parties, be sure to include these instructions for use.

- The device is intended for private use only. The appliance must not be used for commercial or medical purposes.
- If you have any health concerns, consult your doctor before use.
- Use the unit indoors only! Do not use the device in damp rooms (e.g. when bathing or showering).
- Use the device only for its intended purpose according to the instructions for use. In case of misuse, the warranty becomes void.
- If you feel pain or find the massage uncomfortable, stop the application and consult your doctor.
- You must not use the device if you suffer from one or more of the following illnesses or complaints: Circulatory disorders, varicose veins, open wounds, bruises, skin tears, phlebitis or thrombosis.
- Do not use the appliance if you are pregnant. Special care is necessary when the appliance is used on, by or near children, sick and helpless persons.
- Do not use the device near the eyes or other sensitive parts of the body.
- Do not use this device to support or replace medical applications. Chronic conditions and symptoms could worsen.
- Do not use the device while you are sleeping or in bed.
- Do not use the device before falling asleep. The massage has a stimulating effect.
- Do not use the device while driving a vehicle or operating machinery.
- This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product in a safe way by a person responsible for their safety.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must not play with the appliance.
- Cleaning and user maintenance must not be carried out by children without supervision.
- Do not allow the unit to come into contact with pointed or sharp objects.
- Do not cover the unit when it is switched on. Never use it under blankets or pillows. There is a risk of fire, electric shock and injury.
- Do not immerse the unit in water or other liquids.
- If liquid does enter the unit, switch it off immediately and disconnect the power supply if it was connected. Wait until the unit is completely dry.
- Only connect the unit for charging so that the cable and power connector are freely accessible.
- Never leave the unit unattended when it is connected to the mains (for charging) or in operation.
- The built-in batteries are not intended for replacement or removal by the end user. If you experience problems with the power supply, contact customer service.
- Always switch off the appliance after use by pressing button ③.
- To disconnect the appliance from the mains, always pull the plug out of the power socket. Do not pull on the cable!
- Do not carry, pull or twist the unit by the cable or the power plug.
- Make sure that the cable does not pose a trip hazard. Do not bend, pinch or twist the cable.
- Do not use the appliance if any damage is visible, if it is not working properly, if it has been dropped or immersed in water. To avoid hazards, send the unit to the service centre for repair.
- Do not insert any objects into the openings of the unit.
- You may only clean the unit. In case of malfunctions, please do not repair the unit yourself, as this will automatically void all warranty claims. Contact your specialist dealer and have the repairs carried out by the authorised service.
- Only use accessories intended for the unit and supplied by the manufacturer.
- Excessive use can lead to overheating. After 15 minutes of use, allow the appliance to cool down for at least 15 minutes before using it again. The appliance switches off automatically after 15 minutes of use.

**ATTENTION**  
Make sure that the maximum operating time of 15 minutes is not exceeded!

## Scope of delivery

Please first check whether the scope of delivery is complete and that the item is not damaged in any way. If in doubt, do not operate the unit and contact your dealer or service centre.

The scope of delivery includes:

- 1 medisana Massage Gun with Hot & Cold function **MG 600**
- 7 Massage heads
- 1 Quick Reference Guide
- 1 USB charging cable
- 2 replacement rubber rings
- 1 Storage case

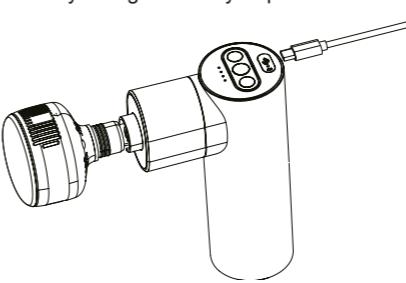
Packaging is recyclable or can be returned to the raw material cycle. Please dispose of packaging material that is no longer required properly. If you notice any transport damage when unpacking, please contact your dealer immediately.

## Device and controls


- 1 Control panel
- 2 Massage head "Hot and Cold"
- 3 On/off button for vibration
- 4 USB charging port
- 5 On/off button Cold function
- 6 On/off button Hot function
- 7 LED charge status indicator
- 8 Battery compartment
- 9 Massage head "Conical"
- 10 Massage head "Ball"
- 11 Massage head "Spade"
- 12 Massage head "Flat"
- 13 Massage head "Spherical"
- 14 Massage head "U"

## Charging the battery

- Before first use, the battery should be fully charged (charging time approx. 5 hours).
- Connect the supplied USB cable to the charging port ④ and a suitable power source (5V, e.g. USB socket or notebook USB port etc.).
- The charge level indicator ⑦ starts flashing. When the battery is fully charged, the charge level indicator ⑦ lights up constantly.
- The average running time of the unit with a fully charged battery depends on the use and is approx. 2 - 3 hours. We recommend that you never allow the battery to run down completely, but that you fully charge the battery again when there is approx. 25% remaining charge.
- The current charge status of the battery can be displayed by briefly pressing any button when the unit is switched off. Each of the 4 LEDs of the charge level indicator ⑦ stands for approx. 25%.

## Fitting / changing the massage head

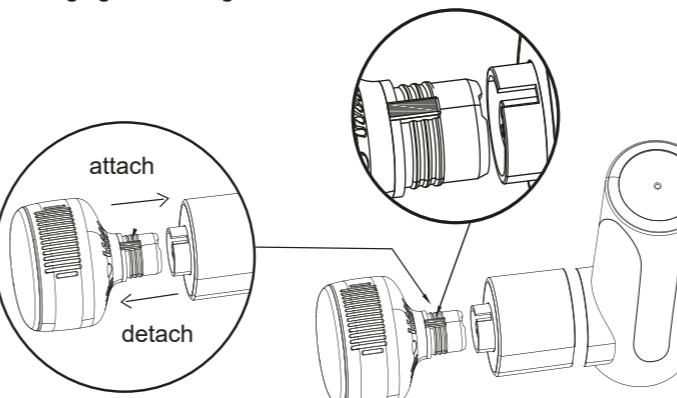- The massage heads are fastened to the main unit using the "tongue and groove" principle (see illustration). Please check the connection of the components before the unit is put into operation.

## Basic operation

- Press and hold button ③ to switch the vibration on or off. After switching on, the LED of the key indicates the set level: Blue = level 1 (low), Purple = level 2 (medium), Red = level 3 (high). You can change the level setting by briefly pressing key ③.
- After switching off (manually or by automatic switch-off after 15 minutes of operation), the charge level indicator ⑦ lights up for approx. 7 seconds and the unit is in standby mode. After another 5 seconds without further operation, the unit switches off automatically.

## Using "Hot & Cold" Functions

To use the "Hot & Cold" functions, the "Hot and Cold" massage attachment (②) must be used! The other massage attachments are NOT suitable for the "Hot and Cold" functions!

**HOT Massage:** This massage is particularly suitable for warming up and preventing muscle strains before exercise, as well as for muscle relaxation and promoting blood circulation. Press and hold button ⑤ to activate the function. After activation, the button's LED indicates the set heat level: blue = level 1 (low), purple = level 2 (medium), red = level 3 (high). You can change the level setting by briefly pressing button ⑤. Press and hold button ⑥ to deactivate the function again.

**COLD Massage:** This massage is particularly suitable for anti-inflammation and for releasing muscle tension and cramps (e.g. after workout). Press and hold button ⑤ to activate the function. After activation, the key's LED indicates the set cooling level: blue = level 1 (lowest temperature), purple = level 2 (medium temperature), red = level 3 (highest temperature). You can change the level setting by briefly pressing button ⑤. Press and hold the button ⑤ to deactivate the function again.

## What are the other massage heads recommended for?

**Massage head ⑨:** Suitable for use in specific areas (trigger points, small muscle groups)

**Massage head ⑩:** Suitable for large muscle groups

**Massage head ⑪:** Suitable especially for arm and leg muscles

**Massage head ⑫:** Suitable for deep massage of the musculature / for full body relaxation

**Massage head ⑬:** Suitable for relaxation massage / loosening of large muscles

**Massage head ⑭:** Suitable for muscles in the area of the spine

## Cleaning and care

- Before cleaning the item, disconnect the power supply if necessary and allow the appliance to cool for at least 15 minutes.
- Clean the unit with a soft cloth slightly moistened with water. Never use harsh cleaning agents, petrol, thinner or alcohol.
- Then dry the unit with a dry, clean cloth.
- Do not expose the unit to direct sunlight, protect it from dirt and moisture.
- Make sure the unit is completely dry before using it again.
- Store the unit in a clean and dry place.

## Disposal

This appliance must not be disposed of with household waste. Every consumer is obliged to hand in all electrical or electronic devices, regardless of whether they contain pollutants or not, to a collection point in his or her city or to a retailer so that they can be disposed of in an environmentally friendly manner. Remove the battery before disposing of the device.

Do not dispose of used batteries/battery packs in household waste, but in hazardous waste or at a battery collection point in a specialist shop. Contact your local authority or dealer regarding disposal.

## Technical specifications

Name and model:

**medisana** Massage Gun with Hot & Cold function **MG 600**

Input: 5V / 2A; Output: 11.1V / 2,400 mAh

after 15 minutes

1: 1,800 rpm / 2: 2,000 rpm / 3: 2,400 rpm

Vibration levels (+/- 5%):

Operating/charging conditions: Use / charge indoors only,

temperature 0°C - 40°C (32°F - 104°F)

Storage conditions: clean, cool and dry

Dimensions: approx. 16 x 14 x 7 cm

Weight: approx. 765 g (incl. cable and massage heads)

Article number: 88987

EAN number: 40 15588 88987 5

**In the interest of continued product improvement, we reserve the right to make technical and design changes.**

The current version of these user instructions can be found at [www.medisana.com](http://www.medisana.com)

## Warranty and repair conditions

Your statutory warranty rights are not restricted by our guarantee below. In case of warranty please contact your specialist shop or the service centre directly. Should you have to return the device, please state the defect and enclose a copy of the purchase receipt or invoice. The following warranty conditions apply:

- All **medisana** products are guaranteed for 3 years from the date of purchase. In case of a warranty claim, the purchase receipt or invoice must be provided as proof of purchase date.
- Faults due to material or manufacturing defects will be rectified free of charge within the warranty period.
- A warranty service does not extend the warranty period for either the equipment or for replacement components.
- The following are excluded from the warranty:
  - any damage caused by improper use, e.g. not following the user instructions.
  - damage caused by repair or intervention by the purchaser or unauthorised third parties.
  - transport damage that has occurred on the way from the manufacturer to the consumer or has arisen during transit to the service point.
  - spare parts that are subject to normal wear and tear.
- Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty claim.

**medisana** GmbH, Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY

The service centre address is shown on the attached leaflet.

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**medisana GmbH**